

'Trust Shipping' is the practice name for 'Trusted Shipping Limited'

# Trust Shipping

Hong Kong

T: +852 5976 2338

E: [service@trustshipping.org](mailto:service@trustshipping.org)

U: [www.trustshipping.org](http://www.trustshipping.org)



## Disposable Breath Alco Analyser

Trust Shipping, as agents, is proud to introduce you, to our 'Disposable Breath Alcohol Tester' Self-Testing Breathalyzer.

The Alcotest Breathalyzer is FDA and MADD Canada approved. Additionally, it is certified according to the standards of TÜV, a certification and technical organization present in 69 countries in Europe and in 32 areas of activities and has met the European standard NF X 20-702.

The Alcotest Breathalyzer is an Inexpensive, Disposable, Certified Breath testing device which quickly and simply provides 4 levels of BAC readings to help determine the level of alcohol. Within 2 to 4 minutes, you obtain a precise result.

### Features

- Dimensions: 10 cm length x 1 cm diameter (diameter of a pencil)
- Press both ends of tube firmly to break seal and blow—it is that easy!
- 4 levels of BAC % readings ( 0.0% – 0.02% – 0.05% – 0.08% )
- Disposable after one use.
- Weight: 6.0 grams
- Duration of life: approximately 24 months



**Easy to use:** Firmly push both ends of the tube to break the security foil, blow for 10 seconds in the direction of the arrow and wait for 2 to 4 minutes to interpret the result. Depending on your level of alcohol intoxication, the crystals will be white (0.00%), Pale pink (0.02%), medium pink (0.05%) or purple (0.08%)

### Instruction for testing using Alcotest Breathalyzer:

Wait 20 minutes after your last drink.

**1**

Firmly press both ends of the tube to break the security foil.

**2**

Take a deep breath and exhale into the tube for 12 seconds in the direction of the arrow marking.

**3**

Test crystals

Scale

Wait 2 to 4 minutes, then compare the colour of the test crystals with the scale to estimate the breath alcohol concentration.